


TAP PAP CS Instructions

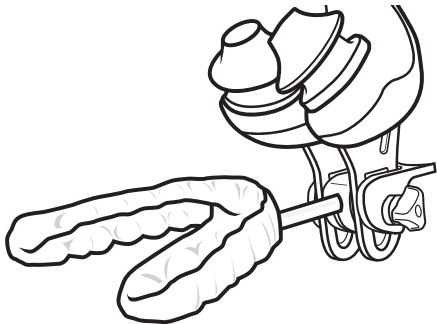

TAP PAP

myTAP™



AM
Aligner.

DENTAL INSTITUTE
OF SLEEP MEDICINE

Intended Use

The TAP PAP CS is designed to connect the TAP 3 or TAP 3 Elite oral appliance to the TAP PAP Nasal Pillow Mask to provide combination Oral Appliance and CPAP/Bi-level therapy. It is for single patient use in the home, hospital, sleep lab or other clinical setting.

WARNINGS:

Read all instructions pertaining to the TAP PAP Nasal Pillow Mask and the TAP 3 or 3 Elite Oral Appliance. Heed all warnings.


The TAP PAP CS should not be used with any other products besides the TAP PAP Nasal Pillow Mask and the TAP 3 or 3 Elite Oral Appliance.

Cleaning Instructions:


Wash the TAP PAP CS with pure soap in lukewarm water for no longer than 10 minutes. Brush the CS Connector with toothbrush, rinse and let air dry.

Parts and Tools for the TAP PAP CS Connector

TAP PAP NPM CS Attachment

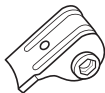

Front Plate with Post

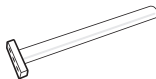


Items that are **included** with the CS Connector but are not pictured:

- Short Hex Key
- Long Hex Key

Items that are **not included** with the CS Connector but will be needed for full assembly of the TAP PAP CS.


- TAP PAP Nasal Pillow Mask-Frame Only (Includes Nut and Bolt)
- TAP 3 or 3 Elite Appliance

Assembling the TAP PAP CS Front Plate with Post to TAP 3 or 3 Elite Upper Appliance

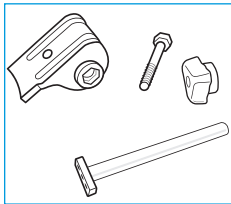


1.

Be sure to have your TAP 3/3 Elite Appliance, your TAP PAP NP Mask, and the CS Connector and Front Plate with Post. (The TAP PAP Nasal Pillow Mask comes with the Nut and Bolt pictured above.)

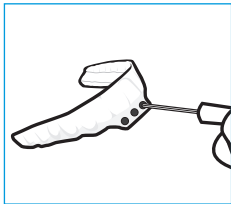


2.

Remove the TAP 3 or 3 Elite Front Plate by removing the 2 Locking Screws (outer screws).

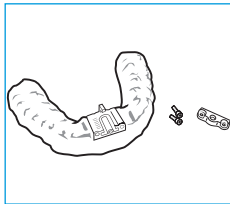

3.


Remove the Front Plate, leaving the hook attached. (Set the TAP 3 or 3 Elite Front Plate aside, this part will no longer be needed.)

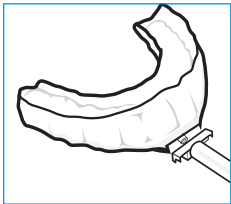

4.

Attach the Front Plate with Post onto the TAP 3 or 3 Elite (where you removed the Front Plate from)


NOTE: Be sure the Front Plate with Post goes on with the flat side facing up.

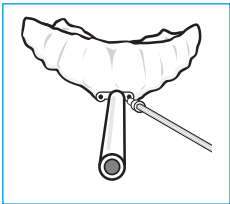

5.

Using the Locking Screws you removed from the TAP 3 or 3 Elite in Step 2, secure the front plate with Post to the TAP 3 Elite Upper Appliance.


Assembling the TAP PAP CS to the TAP PAP Nasal Pillow Mask

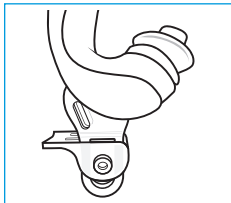

1.


Slide the TAP PAP Nasal Pillow Mask onto the TAP PAP CS.

NOTE: Be sure the flat surface is facing upwards (as shown).

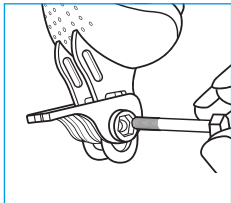


2.

Insert the Nut and Screw into the TAP PAP CS and TAP PAP Nasal Pillow Mask.

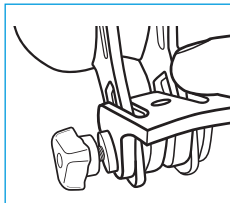


2. Continued

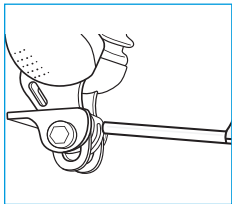

3.

Insert the Front Plate with Post (attached to the TAP 3 Elite Appliance) into the TAP PAP CS (attached to the TAP PAP Nasal Pillow Mask).

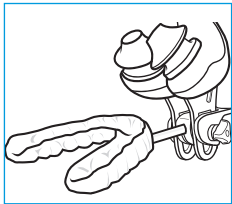

4.

Once the Front Plate with Post is attached to the TAP PAP CS and Nasal Pillow Mask, lightly tighten the Screw and Nut to hold the post into place.

5.

Your TAP PAP CS is fully assembled. Refer to the TAP PAP Nasal Pillow Mask Instructions for fitting the Mask and refer to the TAP 3 or TAP 3 Elite Instructions for the appliance.

AIRWAY MANAGEMENT

3418 Midcourt Road
Suite 114
Carrollton, TX 75006

866.264.7667

AMISLEEP.COM



 facebook.com/airwaymanagement

TAP, myTAP, TAP PAP and **THEMRACRYL** are trademarks of Airway Technologies, LLC, registered in the U.S. and other countries. All rights reserved. 70015 REV A